## EN Eye drops 0.1% HA

These eye drops offer additional comfort and lubrication for mild eye irritation. Suitable for use in combination with all types of contact lenses.

### Application:

These eye drops based on HA provide comfort and hydration. Use of a computer and/or air conditioning can lead to your eyes feeling dry. These eye drops provide additional comfort. Suitable for use with contact lenses (hard and soft).

The easy-to-use special designed squeeze bottle and dropper tip allows for precise dosage of the unpreserved 0.1% HA that provides comfort and lubrication for mild irritation of the eyes.

### Composition:

HA 0.1%, SODIUM CHLORIDE, EDTA, PHOSPHATE BUFFER, PURIFIED WATER. Preservative free.

### Precautions and instructions for use:

Wash and dry your hands before use.  
Do not use when tamper-evident seal is broken before the first time use. Remove the tamper-evident seal <see image 1>.  
Remove the cap from the bottle <see image 2> and lean your head slightly backwards.  
Use your thumb and forefinger to hold one eye open <see image 3>.  
Gently squeeze one or two drops into your eye and blink several times <see image 4>.  
Repeat the process in your other eye.  
Blink several times so the eye drops are distributed over the surface of the eyes.  
Do not touch the dropper tip of the bottle as this may contaminate the solution.  
Close the bottle after use, prevent squeezing at closing <see image 5>.  
Eye drops can also be applied directly to the contact lenses prior to insertion.  
Blink your eyes several times afterwards in order for the drops to cover the eyes.  
When used as instructed, the eye drops can be used safely and whenever needed.

**Storage conditions:**  
Use within 12 weeks after opening (see image A).  
Keep out of reach of children (see image B).  
Do not use one bottle for several people.

### Warnings and contra-indications:

Do not use the eye drops if you are allergic to one of the ingredients.  
Do not use if you have an eye infection.  
Stop using the product immediately if your eye irritation worsens or you experience eye discomfort such as pain or a burning sensation. This can be caused by an allergic reaction of one of the ingredients.  
Do not use in combination with other eye medication before consulting your eye specialist or doctor.  
Do not use when the packaging is damaged (see image C).  
Do not use after the expiry date.  
Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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