EN Isotonic Sodium Chloride 0,9% solution

Isotonic solution for rinsing.
For all types of contact lenses.

Compositie:

SODIUM CHLORIDE 0,9%, PURIFIED WATER.
Preservative free.

Precautions and instructions for use

The rinsing solution can be used in combination with contact lens care solutions as part of your contact lens care regimen.
Wash and dry your hands before use.
Open the ampoule by turning the closure in order to break the tamper-evident seal.
Rinsing: place the lens in the palm of your hand and rinse it with this solution.
Do not touch the dropper tip of the ampoule.

Storage conditions

Keep out of reach of children. (See image A)

Warnings and contra-indications

The rinsing solutions is not an alternative for disinfection or storage solution, always use a multipurpose contact lens solution or disinfection solution in combination.
Do not use if you are allergic to one of the ingredients.
Do not change your contact lens care system without consulting your contact lens specialist.
Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor.
Do not use in combination with other eye medication before consulting your eye specialist or doctor.
Do not use when the packaging is damaged. (See image B)
Do not use after the expiry date.
Always use fresh rinsing solution.
Never use tap water as an alternative for contact lens solution.
Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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Ote Pharma Sol B.V.