## EN Hydrogen Peroxide Solution

For cleaning and disinfecting. For soft contact lenses.

### Composition:

HYDROGEN PEROXIDE, PHENACETINUM CRYSTALLINUM, SODIUM CHLORIDE, PVP K30 EP, PURIFIED WATER.

### Precautions and instructions for use:

This hydrogen peroxide solution should always be used in combination with a sterile neutralising solution and a lens case with air valve. Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel. Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal. Rinse the lens case with a rinsing solution to remove the leftovers. Place the left contact lens in the cup marked "L" and close the cap. Place the right contact lens in the cup marked "R" and close the cap. Fill the lens case with hydrogen peroxide solution up to the fill line. Disinfect your lenses for at least 20 minutes but no longer than 24 hours. After disinfecting the lenses, follow instructions for neutralizing. After a minimum neutralization time of 10 minutes the lenses can be inserted. Do not touch the dropper tip of the bottle as this may contaminate the solution. Close the bottle tightly immediately after use. Replace your lens case frequently to prevent infection due to contamination.

### Storage conditions:

Use within 2 months after opening. Keep out of reach of children (See image 1).

Keep the solution in its original packaging.

### Warnings and contra-indications:

Do not insert your contact lenses directly in your eye without neutralizing your contact lenses since this can lead to pain and burning eyes. Do not use the contact lens solution if you are allergic to one of the ingredients. Always use a lens case with air valve. Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor. If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution. Do not use when the packaging is damaged (See images 2). Do not use after the expiry date. Do not change your contact lens care system without consulting your contact lens specialist. Do not mix or use with other contact lens solution, unless it is indicated in the instructions for use. Never use tap water as an alternative for contact lens solution. Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device. Do not use in an opened eye.

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