EN Neutralising solution

For neutralising, storage and inserting. For soft contact lenses.

Compositie:
CATALASE, GLYCEROL, EDTA, PHOSPHATEBUFFER, PURIFIED WATER.

Precautions and instructions for use:

This sterile neutralising solution should be used in combination with a hydrogen peroxide solution and a lens case with air valve. Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel. Open the ampoule by turning the closure in order to break the tamper-evident seal. Open the lens case and discard the hydrogen peroxide solution. Fill the lens case with neutralising solution and close the lens case. Allow the lenses to neutralize for a minimum of 10 minutes. After neutralising, the lenses may be inserted. Refresh the neutralizing solution after every 24 hours of storage. Discard ampoule after use. Replace your lens case with air valve frequently to prevent infection due to contamination.

Storage conditions:

Keep out of reach of children (See image 1).

Warnings and contra-indications:

Do not insert your contact lenses directly in your eye without neutralizing your contact lenses since this can lead to pain and burning eyes. Do not use the contact lens solution if you are allergic to one of the ingredients. Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor. If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution. Do not use when the packaging is damaged (see image 2). Do not use after the expiry date. Do not change your contact lens care system without consulting your contact lens specialist. Always replace any remaining solution in the lens case with fresh solution. Never use tap water as an alternative for contact lens solution. Always use a lens case with air valve. Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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