EN Eye drop with with Provitamine B5 ampoules

The eye drop protects the eye against complaints such as sensation of dryness and in case of tired, irritated eyes. By applying several drops in the eye, the feeling of discomfort is soothed or entirely reduced.

Suitable for use in combination with all types of contact lenses.

The preservative free uni-dose eye drops with Provitamine B5 is a sterile viscous eye drop. Intended for wetting, soothing, protection and comfort in tired and irritated eyes. The ingredient pro-vitamin B5 promotes and supports the self-regenerating properties of the cornea.

The single use ampoule allows precise dosage of unpreserved 2% Dexpanthenol (Provิตamine B5) eye drop. Dexpanthenol supports in combination with Hydroxypropylcellulose (HPC) eye wetting and supports the recovery and regeneration of tired and irritated eyes.

The ampoules are intended for single use only; discard after use.

Composition:

PROVITAMINE B5 2%, HYDROXYPROPYLCCELLULOSE, SODIUM CHLORIDE, EDTA, PHOSPHATE BUFFER, PURIFIED WATER.

Preservative free.

Precautions and instructions for use

Wash and dry your hands.

Do not use when the packaging is damaged. Open the ampoule by turning the closure until it breaks. Lean your head slightly backwards.

Use your thumb and forefinger to hold one eye open.

Gently squeeze one or two drops into each eye and blink two or three times.

Repeat this process in your other eye.

Blink your eyes several times in order for the drops to cover your eyes.

Eye drops can also be applied directly to the contact lenses prior to insertion. Blink your eyes several times afterwards in order for the drops to cover the eyes.

When used as instructed, the eye drops can be used safely and whenever needed.

Storage conditions

Keep out of reach of children. (See image A)

Warnings and contra-indications

Do not use the eye drops if you are allergic to one of the ingredients.

Do not use if you have an eye infection.

Stop using the product immediately if your eye irritation worsens or you experience eye discomfort such as pain or a burning sensation. This can be caused by an allergic reaction of one of the ingredients.

Do not use in combination with other eye medication before consulting your eye specialist or doctor. Do not use when the packaging is damaged. (See image B)

Do not use after the expiry date.

After application you can suffer from temporary blurred vision, due to the viscosity enhancing ingredients.

Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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