#### EN Eye drop with Dexamphenol

The eye drop protects the eye against complaints such as sensation of dryness and in case of tired, irritated eyes. By applying several drops in the eye, the feeling of discomfort is soothed or entirely reduced. Suitable for use in combination with all types of contact lenses.

This eye drop is a sterile, viscous and clear rewetting and nutritional solution based on pro-vitamin B5. The ingredient pro-vitamin B5 promotes and supports the self-regenerating properties of the cornea.

#### Application:

These moisturising eye drops based on pro-vitamin B5 reduce eye irritation caused by external influences like ambient air, tobacco smoke and pollen. The eye drop can also be applied on soft or rigid contact lenses prior to insertion.

#### Composition:

PHMB 0,0001%, PROVITAMINE B5 2%, HYDROXYMETHYLPROPYLCCELLULOSE, SODIUM CHLORIDE, EDTA, PHOSPHATE BUFFER, PURIFIED WATER.

#### Precautions and instructions for use

Wash and dry your hands before use.

Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal.

Remove the cap from the bottle and lean your head slightly backwards.

Use your thumb and forefinger to hold one eye open.

Gently squeeze one or two drops into each eye and blink two or three times.

Repeat the process in your other eye.

Blink your eyes several times in order for the drops to cover your eyes.

Do not touch the dropper tip of the bottle as this may contaminate the solution.

Close the bottle tightly immediately after use.

#### Storage conditions

Use within 2 months after opening.

Keep out of reach of children. (See image A)

#### Warnings and contra-indications

Do not use the eye drops if you are allergic to one of the ingredients.

Do not use if you suffer from an eye infection.

Stop using the product immediately if your eye irritation worsens or you experience directly after application other discomfort such as pain or a burning sensation and consult your eye specialist or doctor.

Do not use in combination with other eye medication before consulting your eye specialist or doctor.

Do not use when the packaging is damaged. (See image B)

Do not use after the expiry date.

After application you can suffer from temporary blurred vision.

Do not use more than 4 times a day.

Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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