## EN Peroxide Solution

Peroxide solution disinfects, removes protein and stores all soft contact lenses, including silicone hydrogels.

### MEDICAL DEVICE

Please read carefully and keep this package insert for future reference.

**Contents:** Sterile and buffered 3% hydrogen peroxide solution with a phosporic acid stabilizer, sodium chloride and phosphate buffers.

**Directions for use:** Always wash and dry your hands before handling your lenses and after using Peroxide Solution (See image A). Check that the special lens case includes the neutralizing disc (See image B). Place your left and right lenses on the domes inside the appropriate baskets. Close the baskets taking care not to trap the lenses (See image C). Hold the lens case by the top and remove the lenses by pulling them out of the solution. Rinse each, over a sink/basin (See image D). Fill the lens case up to the fill line with fresh Peroxide Solution then immediately place the holder basket into the lens case. Do not invert or shake case (See image E). Your lenses will be ready to wear after soaking for 6 hours (See image F). Once lenses are removed, use the lens case with remaining solution for 2 seconds each, over a sink/basin (See image D). Fill the lens case up to the fill line with fresh Peroxide Solution then immediately place the holder basket into the lens case. Do not invert or shake case (See image E). Your lenses will be ready to wear after soaking for 6 hours (See image F). Once lenses are removed, use the lens case with remaining solution for 2 seconds each, over a sink/basin (See image D). Fill the lens case up to the fill line with fresh Peroxide Solution over a sink/basin, and leave to lid of the case to air dry (See image G).

**Other instructions:** If you do not intend to wear your lenses immediately after the disinfection/neutralization process, repeat the steps in the directions for use after 24 hours before inserting your lenses. If a separate daily cleaner is used, be sure to rinse the lenses thoroughly with saline before soaking in Peroxide Solution.

**Warnings:** Only use the enclosed special lens case and neutralizing disc with Peroxide Solution. **DO NOT USE WITH OTHER LENS CASES** (See image 1) **NEVER** use Peroxide Solution into the eye. **DO NOT** place your lens into the eye until complete neutralization of the solution has occurred (minimum 6 hours) (See image 2). Do not rinse lenses with Peroxide Solution prior to inserting lenses into your eyes. If you want to rinse your lenses, use a sterile saline solution (See image 3).

**Contraindications (reasons not to use):** If you are allergic to any ingredient in this product, do not use. **Failure to discard solution from lens case after use, topping of solution and use of water to care for your lenses may lead to contamination resulting in eye injury and potential loss of vision. See accompanying instructions for additional important safety information.** Do not use Peroxide Solution if the solution is cloudy or has a strong odor. Failure to follow product directions may lead to vision loss or will result in burning and stinging. Peroxide Solution is **NOT** a multi-purpose solution. Do not insert Peroxide Solution or the lens directly into the eye until complete neutralization of the solution has occurred (minimum 6 hours). Do not rinse lenses with Peroxide Solution prior to inserting the lenses into your eyes. If you want to rinse lenses, use a sterile saline solution. Always wash and dry your hands before handling lenses. Do not use tap water, bottled water or saliva with lenses or lens case. Only use fresh solution to clean and disinfect contact lenses. Always replace your solutions, lenses and lens case as described. Do not touch tip of container to any surface. Replace cap after using. Not for use with (thermal) disinfection. Visit your eye care practitioner regularly.

**Precautions:** Always discard solution from lens case after each use. Keep the bottle tightly closed when not in use. Use before expiration date printed on the front and bottom of the container. Do not use after opening or after 35 uses, whichever occurs first. Store at room temperature. Always use solution at room temperature. Do not use if tamper evident seal is missing or broken. Only use solution with specially designed lens case. Do not invert or shake lens case as residual unneutralised peroxide could remain on the lens. Avoid contact of Peroxide Solution with fabrics. Do not hang your contact lens case system without consulting your doctor / contact lens specialist. **KEEP OUT OF REACH OF CHILDREN.**

How supplied: Peroxide Solution is available for purchase in sterile, 120 and 360 ml plastic bottles and supplied with a new peroxide lens case. Bottles and cartons are marked with a lot number and expiration date.

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