## EN Eyelid spray 0,1% HA

This spray offers additional comfort and alleviation for the eyes and soothes eyelids.

### Application:

0.1% HA hydrating eyelid spray is formulated to help improve the comfort of strained, tired eyelids. The spray is rich in hyaluronate, it improves comfort and lubricates the eyelids and surrounding skin. Great for on the go.

It's suitable for everyday use and is safe for use with contact lenses when eyelids are closed.

### Composition:

HYALURONIC ACID 0,1%, SODIUM CHLORIDE, EDTA, PHOSPHATE BUFFER, PURIFIED WATER. Preservative free.

### Precautions and instructions for use

Wash and dry your hands.  
Remove the cap from the spray.  
When using the spray for the first time, press the plunger several times to fill it.  
Spray the product on closed eyelids.  
To create a mist effect, spray at a distance of 10 cm.  
Prevent contact with the nozzle to prevent contamination and damage.  
Place the cap on the spray immediately after use.  
When used as instructed, the spray can be used whenever needed.

### Storage conditions

Use within 12 weeks after opening. (See image A)  
Keep out of reach of children. (See image B)

### Other instructions

When treating children, have an adult do the spraying.

### Warnings and contra-indications

Do not use if you have an eye infection or other eye disease.  
Do not use if you are allergic to one of the ingredients.  
Do not use in combination with other eye medication.  
Stop using the product immediately if you experience irritation of your eyes or eye lids after using the spray.  
Do not use when the packaging is damaged. (See image C)  
Do not use after the expiry date.  
Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.  
Do not use in an opened eye.

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